



LEGACY  
LABORATORY  
SERVICES

# Microbiology Collection Device

## Hemoccult Sensa Slides

The World Leader In Occult Blood Testing		The World Leader In Occult Blood Testing		The World Leader In Occult Blood Testing	
<p><b>Hemoccult II<sup>SM</sup> SENSAS<sup>®</sup></b></p> <p>SENSITIVE SERIAL TEST FOR ENHANCED DETECTION OF FECAL OCCULT BLOOD</p> <p>PATIENT NAME <b>1</b> _____ AGE _____</p> <p>STREET _____</p> <p>CITY _____ STATE _____ ZIP _____</p> <p>SAMPLE COLLECTION DATE _____ PHONE NO. _____</p> <p>PHYSICIAN NAME _____</p> <p>STORE AT CONTROLLED ROOM TEMPERATURE.</p>		<p><b>Hemoccult II<sup>SM</sup> SENSAS<sup>®</sup></b></p> <p>SENSITIVE SERIAL TEST FOR ENHANCED DETECTION OF FECAL OCCULT BLOOD</p> <p>PATIENT NAME <b>2</b> _____ AGE _____</p> <p>STREET _____</p> <p>CITY _____ STATE _____ ZIP _____</p> <p>SAMPLE COLLECTION DATE _____ PHONE NO. _____</p> <p>PHYSICIAN NAME _____</p> <p>STORE AT CONTROLLED ROOM TEMPERATURE.</p>		<p><b>Hemoccult II<sup>SM</sup> SENSAS<sup>®</sup></b></p> <p>SENSITIVE SERIAL TEST FOR ENHANCED DETECTION OF FECAL OCCULT BLOOD</p> <p>PATIENT NAME <b>3</b> _____ AGE _____</p> <p>STREET _____</p> <p>CITY _____ STATE _____ ZIP _____</p> <p>SAMPLE COLLECTION DATE _____ PHONE NO. _____</p> <p>PHYSICIAN NAME _____</p> <p>STORE AT CONTROLLED ROOM TEMPERATURE.</p>	
<p>Open tab only when ready to use</p> <p><b>IVD</b></p>		<p>Open tab only when ready to use</p> <p><b>IVD</b></p>		<p>Open tab only when ready to use</p> <p><b>IVD</b></p>	
<p>U.S. Patent 5,310,680</p> <p><b>Note:</b> Wait 3-5 minutes after sample application, before developing test.</p> <p><b>(1) Develop Test:</b> Open flap and apply two drops of <b>Hemoccult<sup>®</sup> SENSAS<sup>®</sup></b> Developer over each smear. Interpret within 60 seconds.</p> <p><b>(2) Develop Performance Monitor<sup>®</sup> areas:</b> Apply one drop of <b>Hemoccult<sup>®</sup> SENSAS<sup>®</sup></b> Developer between the <b>(+)</b> and <b>(-)</b> Performance Monitor<sup>®</sup> areas. Interpret within 10 seconds.</p> <p> See Product Instructions for details.</p> <p><b>NOT TO BE OPENED BY THE PATIENT</b></p>		<p>Printed in U.S.A. Made in U.S.A.</p> <p><b>Note:</b> Wait 3-5 minutes after sample application, before developing test.</p> <p><b>(1) Develop Test:</b> Open flap and apply two drops of <b>Hemoccult<sup>®</sup> SENSAS<sup>®</sup></b> Developer over each smear. Interpret within 60 seconds.</p> <p><b>(2) Develop Performance Monitor<sup>®</sup> areas:</b> Apply one drop of <b>Hemoccult<sup>®</sup> SENSAS<sup>®</sup></b> Developer between the <b>(+)</b> and <b>(-)</b> Performance Monitor<sup>®</sup> areas. Interpret within 10 seconds.</p> <p> See Product Instructions for details.</p> <p><b>NOT TO BE OPENED BY THE PATIENT</b></p>		<p>© 2010 Beckman Coulter, Inc. May 2010</p> <p><b>Note:</b> Wait 3-5 minutes after sample application, before developing test.</p> <p><b>(1) Develop Test:</b> Open flap and apply two drops of <b>Hemoccult<sup>®</sup> SENSAS<sup>®</sup></b> Developer over each smear. Interpret within 60 seconds.</p> <p><b>(2) Develop Performance Monitor<sup>®</sup> areas:</b> Apply one drop of <b>Hemoccult<sup>®</sup> SENSAS<sup>®</sup></b> Developer between the <b>(+)</b> and <b>(-)</b> Performance Monitor<sup>®</sup> areas. Interpret within 10 seconds.</p> <p> See Product Instructions for details.</p> <p><b>NOT TO BE OPENED BY THE PATIENT</b></p>	
<p>Beckman Coulter, Inc. Brea, CA 92821</p>		<p>Beckman Coulter, Inc. Brea, CA 92821</p>		<p>Beckman Coulter, Inc. Brea, CA 92821</p>	
<p>LOT NO. _____ EXPIRES _____</p>		<p>LOT NO. _____ EXPIRES _____</p>		<p>LOT NO. _____ EXPIRES _____</p>	
<p>51201 4R 08-13</p>		<p>51201 4R 08-13</p>		<p>51201 4R 08-13</p>	

Legacy Laboratory Services 503-413-1234 or toll free 1-877-270-5566

Revised 6/28/2012