

Legacy Laboratory Services

Legacy LabAlert

An Important Update from Legacy Laboratory Services

August 2020

Legacy Laboratory: Leukocyte Alkaline Phosphatase

Effective August 11, 2020, Legacy Laboratory will discontinue performing the Leukocyte Alkaline Phosphatase (LAP) test. All testing will be sent to the ARUP laboratory.

ARUP Laboratories Testing Requirements			
Test to Order	EPIC order	Miscellaneous Lab Order	Comment: ARUP Laboratories, Test Code 0049000, LAP
	Cerner Order	HOLDREF RMT	
	Outreach Interface order	MISC INTF	
Specimen Preparation	<ul style="list-style-type: none">Collect a dark green sodium heparin tube.Draw 5 mL of whole blood (Min. 1 mL). Do NOT centrifuge.Prepare 6 unfixed, well prepared smears.Protect both smears and whole blood from light and pack accordingly to avoid breakage.		
Storage Temperature	Room Temperature		
Transport	<ul style="list-style-type: none">Send Sunday through Thursday only.Whole blood must be received within 24 hours of collectionSmears must be made within 24 hours of collection and received at ARUP within 7 days of collection.		

For additional information, please contact your account representative, client services or consult our website:

Legacy Laboratory Client Services: 503-413-1234, 877-270-5566, www.legacyhealth.org/labservices