



LEGACY
LABORATORY
SERVICES

Legacy Laboratory Services: SARS-CoV-2 Specimen Collection Kit

August 2020

Legacy Laboratory Services is now offering COVID-19 testing to our outreach clients. While inpatients, healthcare workers, and first responders remain a priority, we understand the importance of accurate, timely results for outpatients as well. Clinicians should use their judgment to determine if a patient has signs and symptoms compatible with COVID-19 and whether the patient should be tested.




Step 1: Assemble collection devices

- M4RT Media
- FLOQSwab



Step 2: Collect nasopharyngeal specimen

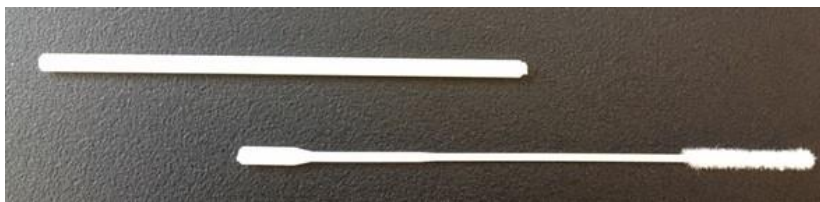

Step 3: Break shaft at scored notch and discard base of shaft



Step 4: Place nasopharyngeal swab in M4RT transport media.

Place tube in zip lock kit baggie, and place baggie in LLS specimen bag.

Refrigerate specimen in LLS specimen bag.